# Flamigel®

## Indicaties

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# Flamigel®

## Indications

**Flamigel®** is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. **Flamigel®** is used post lasertherapy, on superficial burns (including those caused by radiotherapy) and on minor wounds (e.g. grazes, cuts). **Flamigel®** may also be used in superficial open wounds.

## Properties

**Flamigel®** hydro-active colloid formulation can quickly help soothe the pain of minor wounds due to its cooling effect. **Flamigel®** helps to create optimal healing conditions to accelerate cell renewal, allowing the wound to heal fast and so reduce the likelihood of scarring. **Flamigel®** also cushions the wound to provide a barrier against external contamination. **Flamigel®** can be used for dry or wet wounds:

- In case of a dry wound **Flamigel®** turns the dry wound into a moist one ("hydrogel" effect). The dry wound absorbs the water contained in **Flamigel®** and promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, the hydrocolloid of **Flamigel®** is activated and consequently **Flamigel®** absorbs excessive fluid from the wound and restores and maintains the water balance in the wound ("hydrocolloid" effect).

## How to use

Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.

For superficial burns and minor wounds (e.g. grazes, cuts): Carefully apply **Flamigel®** to the wound (without rubbing) once or twice per day (in case of sunburn apply twice to six times daily). If necessary, cover the wound with a plaster or dressing and change daily.

For superficial open wounds: Apply a thick layer of **Flamigel®** (5 mm) to the wound minimising contact with the wound edges. Cover **Flamigel®** with a sterile non-adhesive dressing and fix with gauze or a bandage. Leave **Flamigel®** in place as long as the gel structure is intact. When new skin cells start to form and the wound is closing, only apply a thin layer of **Flamigel®** to prevent weakening. If necessary, protect the wound edges.

- Open wounds: change daily.

## Removing the gel:

- Wash hands before and after removing **Flamigel®**.
- Flamigel®** can be removed simply and painlessly with wet gauze.

## Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use **Flamigel®** when there is a known allergic reaction to parabens or any of the ingredients.
- Do not use **Flamigel®** on the eyelids or in the eyes. If **Flamigel®** comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using **Flamigel®** because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days seek medical advice.

## Storage conditions

- Store at room temperature (< 25°C), in a dry place and in the original packaging.
- Close the tube after use.
- Do not use **Flamigel®** after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year.

## Ingredients

Hydrocolloid, Arginine, Purified water, Macrogol, Branch Chained Fatty Acid (BCFA), Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

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DE: Flen Health GmbH, Königsallee 92A,